

Digital Path (WSI) Consultation at DPALM with EpreDia Scanner

Quick Start

- Case types of consultation: cases can be from UT Outreach, MHH-TMC, LBJ
Consulting may be offered by pathologists at UT office or at Home
- Setting up of scanning slides: H&E and IHC slides of consult cases are scanned with EpreDia P1000 Scanner at UT. Files are stored in [\\isilon.uthouston.edu\EpreDia/users\xxx\yyy](https://isilon.uthouston.edu/EpreDia/users/xxx/yyy)
Where xxx is the faculty name, yyy is the case number
- Set-up of fac's UT office PC:
Installed SlideViewer software (free version by 3DHistech) to read P1000 files
Mapped drive for [\\isilon.uthouston.edu\EpreDia/users](https://isilon.uthouston.edu/EpreDia/users)
- Set-up of fac's Home PC:
Connection to fac's UT office PC with VPN and Remote Desktop
- Steps by pathologists to offer consultation:
Log in to UT office PC (directly at UT or VPN from Home)
Open SlideViewer
Open consult case in [\\isilon.uthouston.edu\EpreDia/users](https://isilon.uthouston.edu/EpreDia/users) to view slides
Communicate with the pathologist who asks for consultation to offer consultation

+++++++